Instruction for Use

Ambu® aView™ 2 Advance

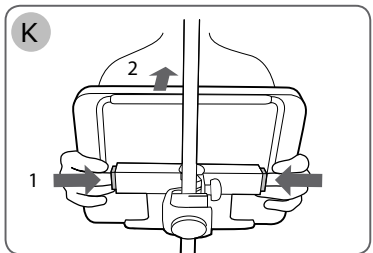
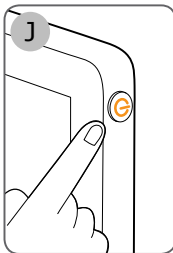
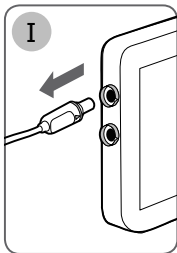
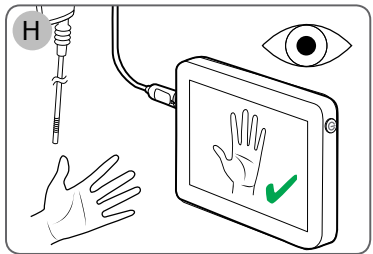
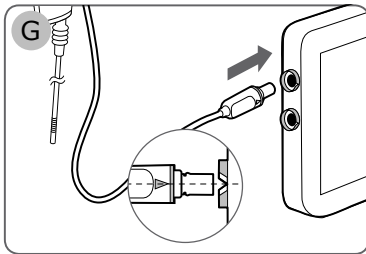
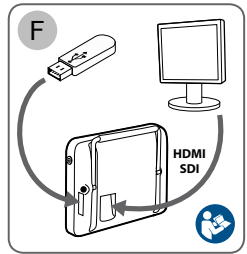
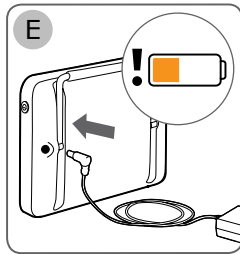
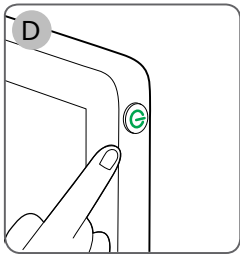
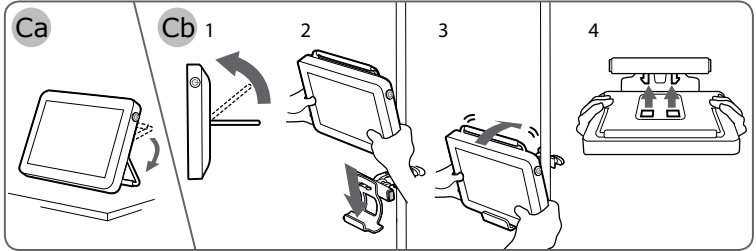
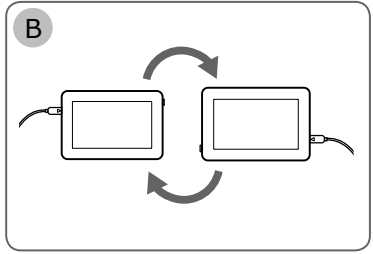
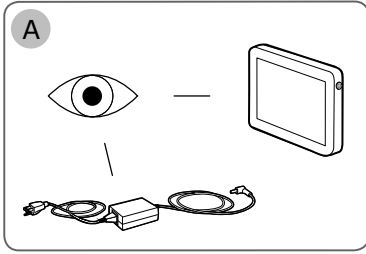
For use by trained clinicians/physicians only.

For in-hospital use.

For use with Ambu® visualization devices.

Further instruction details are available online: www.ambu.com.

Contents

Page

English (Instruction for use).....	4 – 23
Български (Инструкция за употреба).....	24 – 43
Čeština (návod k použití).....	44 – 63
Dansk (brugsvejledning).....	64 – 83
Deutsch (Bedienungsanleitung).....	84 –103
Ελληνικά (Οδηγίες χρήσης).....	104- 123
Español (Instrucciones de uso).....	124 - 143
Eesti (Kasutusjuhend).....	144 - 163
Käyttöohje (suomi).....	164 - 181
Français (Mode d'emploi).....	182 - 201
Hrvatski (Upute za upotrebu).....	202 - 221
Magyar (Használati utasítás).....	222–241
Italiano (Istruzioni per l'uso).....	242-261
日本語 (使用法).....	262-281
Lietuviškai (naudojimo instrukcija).....	282-301
Latviešu val. (Lietošanas pamācība).....	302-321
Nederlands (Gebruiksaanwijzing).....	322-341
Norsk (Bruksanvisning).....	342-361
PL (Instrukcja obsługi).....	362-381
Português (Instruções de utilização).....	382-401
Română (Instrucțiuni de utilizare).....	402-421
Русский (инструкция по применению).....	422-443
Slovensky (Návod na použitie).....	444-463
Slovenščina (Navodila za uporabo).....	464-483
Svenska (bruksanvisning).....	485-503
Türkçe (Kullanma Talimatı).....	504-523
中文 (使用说明).....	524-539

Ambu is a registered trademark and aScope and aView are trademarks of Ambu A/S.

1. Important Information – Read Before Use

Read these safety instructions carefully before using the Ambu® aView™ 2 Advance. The *Instruction for Use* may be updated without further notice. Copies of the current version are available upon request. The latest version of the Instructions for Use is available on www.ambu.com. Please be aware that these instructions do not explain or discuss clinical procedures. They describe only the basic operation and precautions related to the operation of the aView 2 Advance.

In this *Instruction for Use*, the term displaying unit refers to aView 2 Advance. This *Instruction for Use* applies to the displaying unit. For information on a specific Ambu visualization device refer to the relevant *Instruction for Use*.

According to the Spaulding's/CDC classification, the displaying unit including the mounting bracket is a non-critical device.

1.1. Intended Use

The Ambu aView 2 Advance is intended to display live imaging data from compatible Ambu visualization devices.

1.2. Indications for Use

Please refer to the Instructions for Use of the compatible Ambu visualization devices for information on the indications for use.

1.3. Contra Indications

None known for the displaying unit.

1.4. Warnings and Cautions

Failure to observe these warnings and cautions may result in patient injury or damage to the equipment. **Ambu is not responsible for any damage to the system or patient injury resulting from incorrect use.**

WARNINGS

1. Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
2. Be careful to check whether the image on the screen is a live image or a recorded image and verify that the orientation of the image is as expected.
3. To minimize risk of contamination, clean and disinfect the displaying unit after each use in accordance with section 5.
4. Do not use the displaying unit if it is damaged in any way or if any part of the functional check described in section 4.2 fails.
5. Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the system, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
6. To avoid risk of electrical shock only connect ancillary equipment, which are approved as medical electrical equipment.

7. To avoid risk of electric shock, this equipment shall only be connected to a supply main with protective earth.
8. Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
9. To avoid risk of electric shock do not touch the aView 2 Advance stand, VESA mount or video out connectors while simultaneously handling the patient.
10. Only use spare parts supplied by Ambu. Do not modify the spare parts.

CAUTIONS

1. Have a suitable backup system readily available for immediate use so the procedure can be continued if a malfunction should occur.
2. To prevent damaging the device, do not cover the ventilation holes. This could lead to a high surface temperature.
3. Using high frequency tools in a connected visualization device may affect the live image. This is not a malfunction. Wait a few seconds for image to reset.

If as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

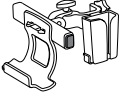
1.5. Adverse Events

None known for the displaying unit.


2. System Description

The displaying unit is reusable. It can be connected to a range of Ambu visualization devices to display video images. (see section 2.2 for compatible devices).

2.1. Displaying Unit Parts

Ambu® aView™ 2 Advance	Item number:	Bracket (e.g. for attaching the monitor to an I.V. pole)	Item number:
	405011000 (SW versions v1.XX)		405013100

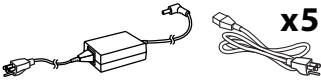For aView 2 Advance model no., please check the label on the back of the displaying unit.
aView 2 Advance is not available in all countries. Please contact your local Ambu representative.





Power supplies		Item numbers:
 x5	Ambu® aView™ 2 Advance power supply manufacturer: FSP Group Inc. Ambu® aView™ 2 Advance power supply part number: FSP065M-DBA	EU/CH 405015300 (not DK & UK) UK 405015400 DK 405015500 AUS & NZ 405015100 US 405015200

2.2. Compatible Devices

Compatible Ambu visualization devices
Ambu® aScope™ 4 Broncho
Ambu® aScope™ 4 RhinoLaryngo
Ambu® aScope™ 4 Cysto
Ambu® VivaSight™ 2 DLT
Ambu® VivaSight™ 2 SLT

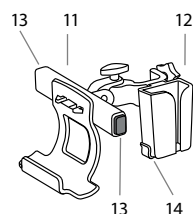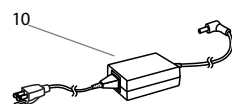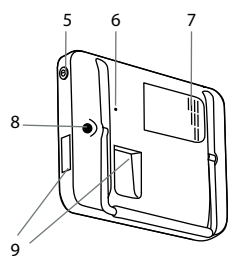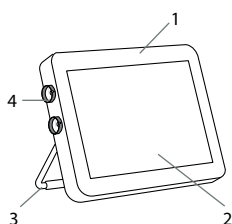The listed devices may not be available in all countries. Please contact your local Ambu representative. Please refer to the *Instruction for Use* of the relevant visualization device.


2.3. Device Description

No.	Part	Function
1	Casing	-
2	LCD Touch Screen	Displays the image from the Ambu visualization device and the graphical user interface.
3	Stand	Use the stand to place the displaying unit on a solid surface and to carry the displaying unit while turned OFF.
4	Connector ports for Ambu® visualization devices	Match and align colours and arrows on device plug and the connector port.
5	Power button	Push button for power ON and OFF.
6	Hardware reset button	Reset the displaying unit hardware without impacting stored data.
7	Ventilation holes	Cools hardware during use.
8	Power inlet	Power inlet for charging the displaying unit.
9	Input/output connections*	USB, HDMI, SDI, LAN *Wi-Fi available.
10	Power supply	Powers the displaying unit. Power cord with country-specific plug.
11	Bracket	Secures the displaying unit to e.g. an IV pole.
12	Power supply bracket	Secures the placement of the power supply.
13	Release buttons	Releases displaying unit from the bracket.
14	Hook	Holder for pouches containing visualization devices.


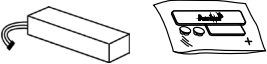
2.4. Spare Parts

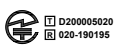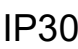Spare parts are intended as replacements for components that are exposed to wear and tear during the lifetime of the device. Consult the troubleshooting guide in section 8 for issues that might require a change of spare parts. In addition to the list below, Ambu® aView™ 2 Advance - Power Supply and Ambu® aView™ 2 Advance – Bracket described in section 2.1 are available as spare parts.

Spare Part		Item numbers:
	Ambu® aView™ 2 Advance - Battery Kit This kit contains a battery and a replacement part kit.	405012100
	Ambu® aView™ 2 Advance - Visualization device interface kit - Blue This kit contains a visualization device interface board, a colour ring and a replacement part kit.	405014100
	Ambu® aView™ 2 Advance - Fan Kit This kit contains a fan and a replacement part kit.	405016100

The spare parts are not available in all countries. Please contact your local Ambu representative.

3. Explanation of Symbols Used

Symbols for the displaying unit	Indication	Symbols for the displaying unit	Indication
	Australia and New Zealand's Regulatory Compliance Mark.		Medical Device.
	Follow the instructions for use.		Country of Manufacturer.
	Japan Radio Law; TELEC RF certification.		Protection against solid objects.
	Waste Bin symbol, indicating that waste must be collected according to local regulation and collection schemes for disposal of electronic and electrical waste (WEEE).		Waste Bin symbol, indicating that waste must be collected according to local regulation and collection schemes for disposal of batteries.
	Universal Serial Bus (USB).		High-Definition Multimedia Interface.


SDI	Serial Digital Interface.	LAN	Local Area Network.
	Humidity limitation.		Atmospheric pressure limitation.
	Medical – general medical equipment as to electrical shock, fire and mechanical hazards only in accordance with ANSI/AAMI E560601-1:2005+ AMD 2012, CAN/CSA-C22.2 No. 60601-1:14+IEC 60601-2-18:2009.		

4. Use of the Displaying Unit

The letters in grey circles  refer to illustrations on page 2.


4.1. First Time System Setup

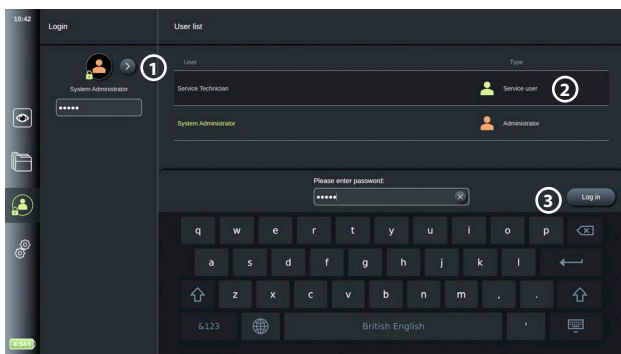1. Unpack the displaying unit and make sure that all parts are present. Refer to the parts described in section 2.
2. Closely examine the displaying unit and other parts for any damage. Do not use the displaying unit if it is damaged in any way **A**.
3. Turn the displaying unit ON by pressing the power button  **D**.
4. Connect the power supply to a power outlet and insert the power plug into the power inlet on the back of the displaying unit **E**.
5. Log in as **Administrator** to get access to system settings. Press the **Login**  tab on the tool bar on the left side of the screen. Press the **arrow** **1** and select the **Administrator** **2** from the **User list**. Enter password and press **Login** **3**. The factory default Administrator password is AmbuAdmin. It is required to change the password. For instructions on changing the password please see the online Reference Manual on www.ambu.com.


6. Change **System language** by pressing the **Settings**  tab in the tool bar on the left. Press **Setup** and select **Language** **4** to open the menu. Open the **System language** **5**

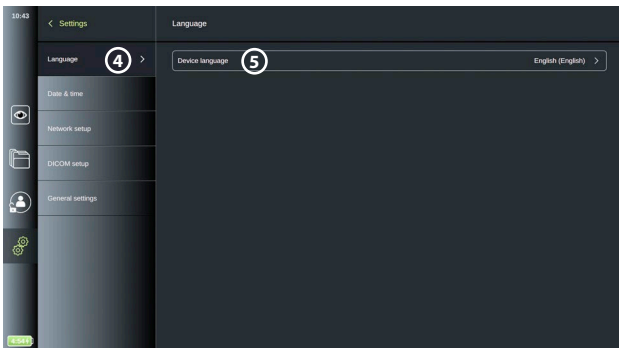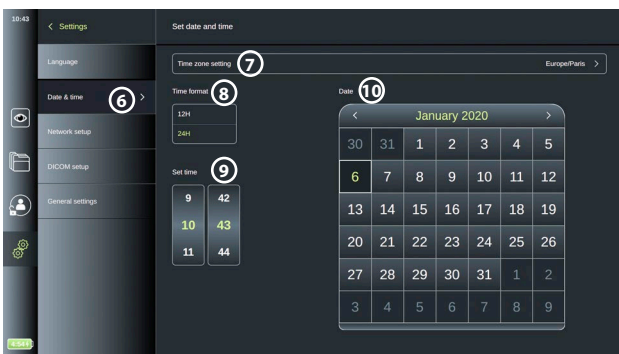drop-down menu and select language from the list. Note that the language will change immediately when selected.


7. Press **Date & Time** (6) to adjust time, date and time zone. Open the **Time zone** (7) drop-down menu and select your time zone setting based on the correct city from the list. Select **Time format** (8), adjust **Set time** (9) and select the current date in the **Date** (10) calendar.

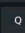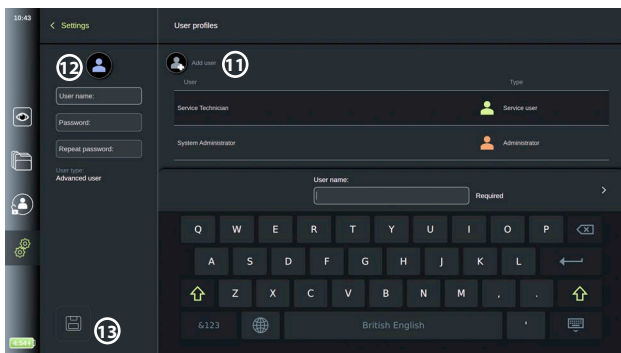
8. For setup of **Network**, **DICOM** and **General Settings** refer to the online Reference Manual on www.ambu.com. Find the Reference Manual in the download section on the product page. Here you will find information regarding cybersecurity. This is important in order to setup and use your product and data, so you are protected from being compromised in terms of Confidentiality, Integrity and Accessibility.

Create User Profiles

As default the **Archive**  is password protected. The Archive can be accessed by the Administrator but it is recommended to create an Advanced User profile.

9. Press the **Settings**  tab in the tool bar on the left. Press **User Profiles** and press **Add user**  to create a new profile. Enter Name, Password, Repeat Password  and press the save button . For more information refer to the online Reference Manual on www.ambu.com.

Description of User Types


User Types and System Access					
User Type		Default User	Advanced User	Administrator	Service User
		Instant access without login	Daily operation	Administrator with full access	Service related tasks
Function Access					
	Login Required		X	X	X
	Live View	X	X	X	X
	Video Recording	X	X	X	X
	Snapshot	X	X	X	X
	Current Procedure	X	X	X	X
	Image Adjustments	X	X	X	X
	Archive	X*	X	X	
	Settings			X	X**

*Access to Archive without log in can be enabled by Administrator in Settings.
 **Service User does not have permission to add and change user profiles.

4.2. Preparation and Inspection of the Displaying Unit

1. Closely examine the displaying unit and other parts for any damage. Do not use the displaying unit if it is damaged in any way .

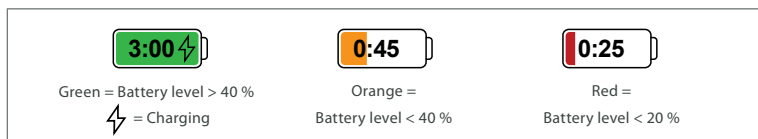2. Clean and disinfect the displaying unit (see section 5).
3. If needed, rotate the displaying unit to have the connector ports for the visualization device oriented left or right in your preferred direction **B**.
4. Choose where to place the displaying unit.
 - a) Place the displaying unit on a solid flat surface by folding out the stand on the back of the displaying unit **Ca**.
 - b) Mount the supplied bracket on a pole. First, fold the stand on the back upwards. Hold the displaying unit with two hands and place the bottom edge in the bracket. Push displaying unit towards the bracket until it clicks in place. Make sure that the bracket and the displaying unit is securely in place by tightening the bolts on the bracket **Cb**.
5. Turn the displaying unit ON by pressing the power button **D**. While the user interface is loading, a live image is available if an Ambu visualization device is connected.
6. Check the battery indicator **E** in the lower left corner of the screen when the user interface appears **E**. The colour of the battery icon indicates the power level. It is recommended to charge the displaying unit if the battery icon is orange or red.

Charge the displaying unit if necessary by connecting the power supply to a mains socket and inserting the power plug into the power inlet on the back of the displaying unit. A lightning icon ⚡ is shown in the battery indicator on the screen when the displaying unit is charging. Make sure the power supply is present and working at any time. It is recommended to locate the nearest power outlet before starting any procedure.

7. An external monitor can be connected to the HDMI or SDI port located in the back of the displaying unit. USB ports for file export or software updates are placed on the side (see section 2.3). Please consult the manual of the external equipment for further information. **F**

To avoid risk of electrical shock only connect ancillary equipment, which is approved as medical electrical equipment.

8. Connect the Ambu visualization device to the displaying unit by plugging the cable connector into the corresponding connector port. Make sure that the colours are matching and the arrows are aligned **G**.
9. Verify that a live video image appears on the screen, by pointing the distal end of the Ambu visualization device towards an object, e.g. the palm of your hand. Make sure that the live image has the correct orientation **H**.

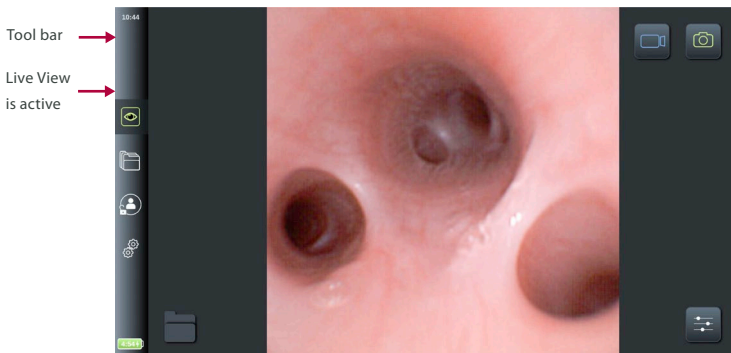
4.3. Installing, Preparing and Operating the Ambu Visualization Device

Please refer to the *Instruction for Use* for the specific visualization device.

4.4. Operating the Displaying Unit

Live View

When the user interface appears the **Live View**  tab is active and a live image is shown. The active tab in the **tool bar** is always marked with **green**. The **tool bar** is always located in the left side of the interface.

Explanation of icons in **Live View**



Icon	Name	Function
	Live View	This tab shows the live image from the connected Ambu visualization device.
	Video Recording	Press the blue Video Recording button to start recording video during the procedure. Press again to stop recording.
	Snapshot	Press the green Snapshot button to take a still image during the procedure.
	Archive	Access recordings. Manage and export files. Login may be required.
	Login	Login is required to access settings, archive and export files with DICOM.
	Settings	Access system settings.
	Battery	The time stamp indicates the time left on the battery while using a visualization device.
	Current Procedure	View videos and snapshots recorded in the current procedure.
	Image Adjustments	Adjust colour, contrast, sharpness and screen brightness.

Dual View

The displaying unit can show two live images simultaneously when two Ambu visualizations devices are connected in at the same time.

The displaying unit will show two live images: a bigger image on the left and a smaller image on the right. The images are not overlapping. In the upper left corner of each live image a roman numeral (I, II, III) is indicating which visualization device is displayed in each image. The roman numerals refer to the corresponding roman numerals placed by each of the connector ports on the side of the displaying unit.

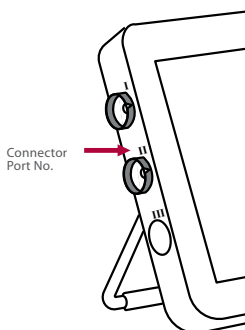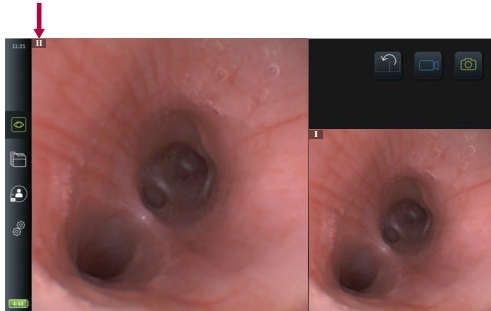Connector Port No.

For further instructions on each function refer to the online Reference Manual on www.ambu.com. Find the Reference Manual in the download section on the product page.

4.5. After Use of the Displaying Unit

1. Disconnect the Ambu visualization device from the displaying unit **I**. For disposal of the visualization device, refer to the *Instruction for Use* of the specific device.
2. Turn the displaying unit OFF by pressing the power button **⏻**. A notification is shown on the screen. Press **OK** to confirm shutdown. This means that the displaying unit is powering down **J**. If the battery level is low, charge the displaying unit (see section 4.2).
3. If needed, dismantle the displaying unit from the bracket. Use two hands to hold the displaying unit, while pressing the two grey release buttons on the bracket behind the displaying unit. Pull the displaying unit towards yourself **K**.
4. Clean and disinfect the displaying unit (see section 5).
5. If the power button **⏻** is flashing orange when the displaying unit is turned OFF, the battery level is below 20%. In this case, charge the displaying unit (see section 4.2).
6. Between procedures, the displaying unit must be stored in accordance with local guidelines.

5. Cleaning and Disinfection of the Displaying Unit

Ensure that the displaying unit is cleaned and disinfected before every use according to good medical practice following the instructions below.

The displaying unit is a reusable medical device. According to the Spaulding's/CDC classification, the displaying unit is a non-critical device.

Cleaning and Disinfection procedure

The displaying unit can be cleaned and disinfected using disposable germicidal hospital disinfection wipes designed for hard, non-porous surfaces. Compatible disinfection wipes are based on Isopropyl alcohol 70-80 %, Isopropyl alcohol/Ammonium Chloride or Isopropyl alcohol/Ethanol.

1. Turn OFF the displaying unit and make sure that it is unplugged from mains.
2. Follow the cleaning instruction from the germicidal disinfection wipe manufacturer to thoroughly clean the displaying unit.
3. Allow the displaying unit to dry after the cleaning.
4. Follow the disinfection instruction from the germicidal disinfection wipe manufacturer to thoroughly disinfect the displaying unit with a new wipe.
5. Allow the displaying unit to dry after the disinfection before use.
6. If the cleaning and disinfection chemicals used leave a residual layer on the display after they dry out, use an alcohol based wipe to remove the residue.
7. If applicable, clean and disinfect the mounting bracket following the same procedure as for the displaying unit.

NOTE: The wipes shall be moist, but not dripping to ensure no damage to internal electronics of the displaying unit.

NOTE: If using wipes containing hypochlorite or citric acid, ensure that all residue is completely removed. Wipes containing hypochlorite or citric acid may affect the screen's antireflective coatings over time. You should limit the use of wipes containing hypochlorite or citric acid to required cases only.

After cleaning and disinfection, the displaying unit must be submitted to the pre-check procedure in section 4.2.

6. Maintenance and Disposal

The displaying unit shall be subject to preventive inspection prior to use (see section 4.2) and shall be cleaned and disinfected according to section 5.

During operation and storage of the displaying unit, the environment must be kept as specified in section 7.2.

No other preventive inspection, maintenance or calibration activities are required for the displaying unit.

6.1. Maintenance of Battery

To prolong battery life it is recommended to fully charge the displaying unit at least every third month. If the battery is discharged the procedure can take up to 3 hours. The battery should be charged at temperatures between 10 - 35 °C.

If the battery needs replacement, only use spare parts supplied by Ambu. Do not modify the spare parts. For more information contact your local Ambu representative.

6.2. Disposal

At the end of product life cycle, clean and disinfect the displaying unit. Before disposing the displaying unit, it is advised to follow these steps to erase all the data:

- Login as an administrator.
- Delete all files stored in the **Archive**. Select all folders and press the trash bin icon.
- Delete Wi-Fi network and password from the **Settings – System setup – Network setup** menu.
- Delete DICOM network from the **Settings – System setup – DICOM setup** menu.
- Delete all Advanced Users from the **Settings – User profiles** menu.

Then, open the displaying unit and dispose the battery and displaying unit separately in accordance with local guidelines.

7. Technical Product Specifications

7.1. Standards Applied

The displaying unit function conforms with:

- IEC 60601-1 Medical electrical equipment – Part 1: General requirements for basic safety and essential performance.
- IEC 60601-1-2 Medical electrical equipment – Part 1-2 General requirements for safety – Collateral standard: Electromagnetic compatibility - Requirements for test.
- IEC 60601-2-18 Medical electrical equipment – Part 2-18: Particular requirements for the basic safety and essential performance of endoscopic equipment

The power supply conforms with:

- IEC 60601-1 Medical electrical equipment – Part 1: General requirements for basic safety and essential performance.

7.2. Specifications for aView 2 Advance

Display

Resolution	1920 x 1080 pixels.
Orientation	Landscape.
Display type	12.8" colour TFT LCD.
Start up time	Immediately after pressing the power button.

Connections

2 x USB connection	USB 3.0 Type A.
Digital video out	HDMI and 3G-SDI (see note). Video format: 1920 x 1080 p, 60 fps.
WIFI	Supports IEEE Standards 802.11ac/a/b/g/n.
LAN	Ethernet RJ45 connector, 10/100/1000 Mbps.

Note: for 3G-SDI interface, it is recommended to use a quality cable with better shielding (eg. RG6 grade).

Memory

Total SSD hard drive size (for storing system software and user image files)	32 GB.
------------------------------------------------------------------------------	--------

Mounting interface

Mounting interface standard	VESA MIS-D, 75 C, VESA FDMI compliant display, Part D, with centre located mounting interface.
-----------------------------	------------------------------------------------------------------------------------------------

Electrical power

Power requirement	19.0 VDC; 3.43 ADC.
Battery type	14.4 VDC 6500 mAh.
Battery Operation	Typical battery runtime of a new, fully charged battery (the displaying unit turned on and scope connected) is more than 3 hours.
Protection against electrical shock	Internally Powered.

Environment conditions

Transportation temperature	Minus 10 - 55 °C (14 - 131 °F).
Storage temperature	10 - 40 °C (50 - 104 °F).
Operation temperature	10 - 35 °C (50 - 95 °F).
Operation relative humidity	30 - 85 %.
Operation atmospheric pressure	80-109 kPa.
Operation altitude	≤ 2000 m.
IP Protection Classification System	The aView 2 Advance is classified IP30. Protection against solid objects.

Dimensions

Width	331 mm (13.03").
Height	215 mm (8.46").
Thickness	52 mm (2.05").
Weight	2.7 kg (6.0 lbs).

7.3. Specifications for aView 2 Advance Power Supply

Dimensions

Weight 360 g (0.79 lbs).

Electrical power

Power requirement 100 - 240 V AC; 50-60 Hz; 1.0-2.0 A.

Power out 19.0 VDC 3.43 ADC.

Protection against electrical shock Class I.

Operating and storage environment

Temperature 10 - 40 °C (50 - 104 °F).

Relative humidity 30 - 85 %.

Power cables

Between the power supply and the displaying unit Ø6.5 mm DC jack connector

Power cables 6 interchangeable types

1. USA and Japan: Model NEMA 5 AC grounded power plug.
2. Australian: AS3112, AC grounded power plug.
3. UK: BS1363, AC grounded power plug.
4. European: CEE 7, AC grounded power plug.
5. Danish: 2-5a, AC grounded power plug.
6. Swiss: Type J, AC grounded power plug (only available as a spare part).

7.4. Specifications for aView 2 Advance Bracket

Bracket

Fits poles with thicknesses Ø 18 - 35 mm (0.7 - 1.5").

Contact Ambu for further information.

8. Troubleshooting

If problems occur with the displaying unit, please use this troubleshooting guide to identify the cause and correct the error. If the problem cannot be resolved by the actions in the troubleshooting guide, contact your local Ambu representative. If replacement of spare parts is necessary, you can find a Spare Part Replacement Manual on www.ambu.com. Here you will also find information about the available Ambu authorized spare parts.

Problem	Possible cause	Recommended action
The displaying unit does not turn ON when the power button is pressed.	No power is left on the displaying unit battery.	Connect the power supply to the displaying unit.
	The displaying unit does not turn ON even though the power supply is connected.	<p>Check that the light on the power supply is ON when it is plugged into a working wall socket. If there is no light, the power supply may need to be replaced.</p> <p>Force the displaying unit to turn OFF</p> <ol style="list-style-type: none"> 1. Press the power button for 10 seconds to force Turn OFF. 2. Press the power button again to turn the displaying unit back on. <p>If this does not work reset displaying unit hardware</p> <ol style="list-style-type: none"> 1. Remove all connected items including the power supply. 2. Press the hardware reset button on the back of the displaying unit*. 3. Reconnect the power supply. 4. Press the power button to turn ON the displaying unit.
The battery is not charging.	The internal temperature in the displaying unit is over 45 °C.	<p>If any of the following conditions apply, correct it to cool down the displaying unit:</p> <ol style="list-style-type: none"> 1. Check if the back of the monitor has free access to air flow. 2. Check that the fan is running by looking through the ventilation holes, feeling for airflow at ventilation holes and/or by listening for fan noise. If the fan is not running, it may need to be replaced. 3. Check if the displaying unit is placed in direct sunlight or is exposed to other heat sources.
Battery uptime is low.	Capacity of internal battery has degraded due to wear and tear.	If you experience that the battery life of the displaying unit after a full charge is below 1 hour, consider replacing the battery.

<p>No Live image from the visualization device on the displaying unit screen</p> <p>or</p> <p>No light in the visualization device LED.</p>	<p>Failure on visualization device or visualization device connection.</p>	<p>Connect/reconnect visualization device.</p>
	<p>The displaying unit and the Ambu visualization device have communication problems.</p>	<p>Restart the displaying unit</p> <ol style="list-style-type: none"> 1. Press the power button to turn OFF the displaying unit. 2. When the displaying unit is OFF, press the power button again to turn it back ON. <p>If the displaying unit does not turn OFF, force it to turn OFF by pressing the power button for 10 seconds. Press the power button again turn the displaying unit back on.</p> <p>If this does not work reset displaying unit hardware</p> <ol style="list-style-type: none"> 1. Remove all connected items including the power supply. 2. Press the hardware reset button on the back of the displaying unit*. 3. Reconnect the power supply. 4. Press the power button to turn it back ON.
	<p>Failure on visualization device connector port.</p>	<p>Try the other visualization device connector port.</p> <p>If an image is obtained from the other connector port, then the first connector port may need to be replaced.</p> <p>If none of the connector ports provide an image, try again using a new visualization device.</p>
<p>A recorded image is shown in the Procedure Folder tab</p> <p>or</p> <p>A menu box is blocking the live image.</p>	<p>Return to live image by pressing the Live View tab or</p> <p>If this does not work restart the displaying unit</p> <ol style="list-style-type: none"> 1. Press the power button to turn OFF the displaying unit. 2. When the displaying unit is OFF, press the power button again to turn it back ON. 	

<p>Live image on the displaying unit screen is only working while booting.</p>	<p>Processing failure.</p>	<p>Force the displaying unit to turn OFF</p> <ol style="list-style-type: none"> 1. Press the power button for 10 seconds to force Turn OFF. 2. Press the power button again to turn the displaying unit back on. <p>If this does not work reset displaying unit hardware</p> <ol style="list-style-type: none"> 1. Remove all connected items including the power supply. 2. Press the hardware reset button on the back of the displaying unit*. 3. Reconnect the power supply. 4. Press the power button to turn it back ON.
<p>No image or User Interface on the external screen.</p>	<p>External screen is turned OFF.</p>	<p>Turn ON external screen.</p>
	<p>Issue with cable or cable connection.</p>	<p>Check that the SDI or HDMI cable is connected properly.</p> <p>If possible, try a new cable. It is recommended to use a high-quality cable with better shielding. (eg. RG6 grade).</p>
	<p>Wrong input channel is selected on the external monitor.</p>	<p>Make sure the correct input channel is selected on the external monitor.</p>
	<p>SDI – External screen does not support 3G-SDI.</p>	<p>Make sure that your external screen supports 3G-SDI (1920 x 1080p 60 fps.).</p>
<p>The colours, contrast, sharpness and brightness look different on the external screen than on the displaying unit screen.</p>	<p>Image settings on external screen are not correct.</p>	<p>Adjust the image settings on the external screen to achieve the desired result.</p> <p>Refer to the <i>Instructions for Use</i> for the external screen.</p>

Touch interface does not respond.	The displaying unit has internal communication problems.	Restart the displaying unit <ol style="list-style-type: none"> 1. Press the power button to turn OFF the displaying unit. 2. When the displaying unit is OFF, press the power button again to turn it back ON.
		If the displaying unit does not turn OFF, force it to turn OFF by pressing the power button for 10 seconds. Press the power button again turn th displaying unit back on.
		If restart and force turn OFF does not work, reset displaying unit hardware <ol style="list-style-type: none"> 1. Remove all connected items including the power supply. 2. Press the hardware reset button on the back of the displaying unit*. 3. Reconnect the power supply. 4. Press the power button to turn it back ON.
Low picture quality.	Dirty / damp screen.	Wipe the screen with a clean cloth as described in section 5.
	Image Adjustment settings are not optimal.	Open the Image Adjustment menu to adjust colour, contrast, sharpness and brightness.
	The distal end in the visualization device is dirty / damp.	Refer to the visualization device <i>Instructions for Use</i> .

* Access the hardware reset button using pin tool with a diameter of 1.5mm and length 16mm.

9. Warranty and Replacement

9.1. Limited Warranty

Ambu warrants that the displaying unit (as defined in section 2.1) will conform to the specifications described by Ambu and be free from defects in material and workmanship for a period of one (1) year from the date of invoice.

Under this limited warranty, Ambu will be responsible only for either supplying authorized spare parts or replacement of the displaying unit, as Ambu may decide in its sole discretion.

In case of replacement of spare parts, the customer is obliged to provide reasonable assistance to Ambu, including, where relevant, by customer's technicians pursuant to instruction from Ambu.

Unless otherwise expressly agreed in writing, this warranty is the only warranty which applies to the displaying unit, and Ambu expressly disclaims any other warranty, expressed or implied, including any warranty of merchantability or fitness for a particular purpose.

The warranty applies only if it can be established that:

- a) The displaying unit has not been disassembled, repaired, tampered with, altered, changed, or modified by persons other than technical personnel (except with the prior written consent of Ambu or in accordance with the instructions in the Spare Part Replacement Manual); and
- b) The defects or damage to the displaying unit does not result from abuse, incorrect use, negligence, improper storage, inadequate maintenance or use of unauthorized accessories, spare parts, consumables, or supplies;

In no event shall Ambu be liable for any indirect, incidental, consequential or special damages of any kind (including without limitation loss of profits or loss of use), whether or not Ambu shall be or should be aware of the possibility of such potential loss or damage.

The warranty applies only to the original customer of Ambu and cannot be assigned or otherwise transferred.

In order to avail itself of this limited warranty, if requested by Ambu, the customer must return the displaying unit to Ambu (at its own expense and risk of shipment). In compliance with applicable regulations any displaying unit that has come into contact with potentially infectious material must be decontaminated before being returned to Ambu under this limited warranty (pursuant to the cleaning and disinfection procedures in Section 5 above). Ambu is entitled to reject a displaying unit which has not been duly decontaminated.

Appendix. Electromagnetic Compatibility and Radio Frequency Compliance

Like other electrical medical equipment, the system requires special precautions to ensure electromagnetic compatibility with other electrical medical devices. To ensure electromagnetic compatibility (EMC) the system must be installed and operated according to the EMC information provided in the Reference Manual.

To ensure Radio Frequency (RF) compliance the system must be installed and operated according to the RF information provided in the Reference Manual.

The Reference Manual can be downloaded on www.ambu.com.

The system has been designed and tested to comply with IEC 60601-1-2 requirements for EMC with other devices. Please observe the following:

Notes: The emissions Characteristics of this equipment makes it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to the radio-frequency communication service. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

The system contains the following radio transmitters compliant with the requirements of Directive 2014/53/EU and FCC part 15:

- Wi-Fi - IEEE802.11ac/a/b/g/n

Hereby, Ambu declares that the radio equipment type aView 2 Advance is in compliance with Directive 2014/53/EU.

The full text of the EU declaration of conformity is available at the following internet address: www.ambu.com.

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